Original

User Manual

Italiano

English

Deutsch

Français

Nederlands

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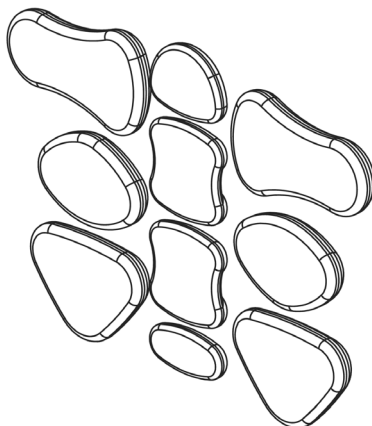
Pad imbottiti

What is the Tarta® system?

The modular TARTA® system is a backrest devised to be adaptable to each individual for improving their posture. Using a combination of design, materials and technology, TARTA® represents the best of ergonomics. Designed by a pool of experts on posture, it is a product that offers a high degree of comfort, using a futuristic and fascinating design and following the “one fits all” philosophy.

The original idea behind the TARTA® system was to improve the posture of people with disabilities, with all their special needs, but we quickly realised that we had to extend the solutions we came up with to other fields of application for the benefit of the many people who spend a great deal of their day seated. Some examples that we have produced in addition to the application for wheelchairs, are office chairs, easy chairs, a babies’ high chair and a handy “mobile system” for settees and armchairs. But this is just the beginning – the applications are really endless.

The materials used in TARTA® have been designed to achieve the highest levels of performance.

The pre-stressed steel used for the springs twists to follow the movement of the person, giving freedom of movement, and then returns to its original position.

The padding is made of washable, breathable material.

The TARTA® backrest consists of a real skeleton similar to the rib cage in which aluminium staves, (that are adjustable) that conform to the position of the person, are attached to central “vertebrae” (that can also be tailored in shape and size) that simulate a spinal column.

The entire system is then finished with padding in various shapes and sizes, to optimise the personalisation.

The system, in its simplicity, provides many benefits both in its standard use and when fitted by expert technicians, providing the possibility of producing customised versions using all kinds of devices that the backrest then hides within its clean and uncluttered lines.

Warnings

- The device must be prescribed only by a specialist doctor.
- The specialist doctor has the responsibility to:
 1. prescribe the device.
 2. prescribe the size of device most suitable for the user.
 3. prescribe to the user the appropriate time for continued usage of the device.
- Should the device not be complete or if it is damaged, do not use it but immediately contact the orthopaedic centre that supplied it.
- The installation of a TARTA® backrest on a wheelchair may affect its centre of gravity and cause it to overturn potentially causing injury. For this reason, after having installed the TARTA® system, the stability of the wheelchair must be checked while it is in motion on an incline of between 12° and 15° in all directions (forwards, backwards, to the left and to the right). Always consider the possible need to equip the wheelchair with anti-tipping wheels or with an amputee plate in order to improve the stability.
- Do not use the TARTA® backrest to push or lift the wheelchair. Incorrect usage may cause the backrest to become detached accidentally from the wheelchair.
- After having carried out the required adjustments, make sure that the supports are sufficiently tight and the backrest is ready for use.
- Periodically check that the mechanical parts that make up the device have not become loose or have become misaligned in comparison to the shape that the device had at the time of acquisition; should such a situation be encountered, have the device adjusted by a technician authorised to carry out maintenance on TARTA® systems.
- The TARTA® system must be regulated and installed only by accredited health workers, orthopaedic technicians or personnel authorised by Tartadesign srl.
- It is advisable for the user's position to be changed periodically (if possible) in order to avoid excessive pressure on any one part of the body, thus making sitting in it more comfortable.
- Avoid contact with naked flames or glowing objects: they could damage the device and cause injury to the user.
- **Avoid exposing your device to direct sunlight. Do not leave the device inside the car when exposed to the sun. The excessive temperature that develops can damage the adhesive applied on the velcro straps slats.**
- Avoid contact with sharp and/or pointed objects or instruments that might damage the physical integrity of the device, especially the padding.
- Before seating the user on the device, check that the temperature of the parts with which he will come into contact is appropriate (neither too hot nor too cold).
- No modifications should be made to the structure or the components of the device by anyone other than a qualified technician.
- For correct installation the wheelchair must be fitted with a tensionable backrest.

The device has been designed to be able to support a user with a maximum weight of 100 kg.

Contraindications

- Do not use in direct contact with the skin; it is advisable for the user always to wear a garment that makes it possible to avoid direct contact with the material of which the device is made in order to avoid the possibility of any allergic reaction.
- Do not use in contact with any injured areas.
- Do not use in the event of intolerance to the materials of which the device is made.

Failure to comply with the instructions provided above could cause serious injury to the user himself or to other people.

Cleaning and maintenance

- The individual parts of the device can be cleaned with a soft cloth soaked in water and well wrung out.
- Do not immerse the device in water.
- Do not carry out maintenance on the device: if damaged, contact your orthopaedic centre.
- The padding can be washed in a washing machine at 30° C.

Symbol key

	CE Mark – Complies with Directive 93/42/EEC and subsequent modifications
	This symbol identifies the manufacturer and must be accompanied by the manufacturer's name and address.
	Product code
	Production batch
	Warning! Before using the device read the instructions for use.
	Latex free
	Danger

Disposal

- The device can be included in solid urban waste complying with local laws on the matter of waste recycling.

Limited 24 month guarantee

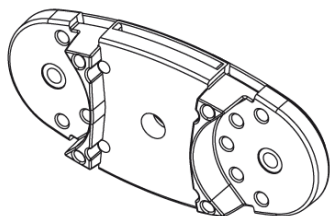
Each TARTA® backrest is carefully checked and tested in order to provide the highest level of performance. Each TARTA® backrest is guaranteed to be free of defects in manufacture and materials for a period of 24 months from the date of purchase, assuming that it has been subjected to normal usage. Should a defect in manufacture or material be discovered within 24 months of the original purchase date, Tartadesign srl undertakes, at its sole discretion, either to repair or to replace the product without charge. This guarantee does not cover holes, rips or burns.

Any complaints or repairs should be referred to the nearest authorised supplier. Except for the guarantees expressly stated herein, all other guarantees are excluded, including all implied guarantees of merchantability or suitability for specific purposes.

Guarantees except as described above are excluded. The application of the guarantee is limited to the repair or replacement of the goods. In no event may injury caused by breakage of the product under guarantee result in claims for consequential damages nor may claims exceed the value of the non-conforming goods sold.

How it is made up

The heart of the mechanism consists of a series of "vertebrae" that, joined together by elements (springs) made of pre-stressed steel, reproduce the shape and characteristics of a spinal column.

Vertebra

Spring 1 mm thick

The column can be made up of a variable number of vertebrae, from a minimum of 3 for the very active version up to the height of the shoulders for persons who require total support of the back.

Each individual vertebra has a height of 5 cm and an excursion of 1 cm, so that the TARTA® system can offer the possibility of varying heights using the same backrest.

On each vertebra there are four plugs which, when removed, enable a lateral inclination of the column of about 10°, where it is necessary to follow the line of significant deformities, such as a scoliosis.

To each of the vertebrae it is possible to fit "staves", that are available in 4/5 different sizes, and that support and/or wrap around the back.

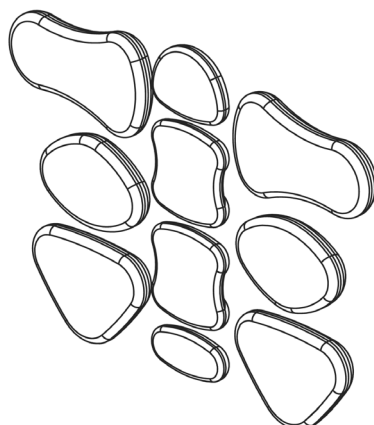
Each of these can be adjusted by simply moving the locking plug in order to find the exact position for containment or support.

The materials used for the staves is aluminium; it has the advantage that it can be modelled millimetrically to meet each individual need.

Staff

Once installed, the padding is fitted and easily adjusted by means of the Velcro fasteners.

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