Rev.03-13.06..2024

The Guide


English
Italiano
Deutsch

Download the
Technical sheet
at:

www.osd.it

or

www.tartadesign.it

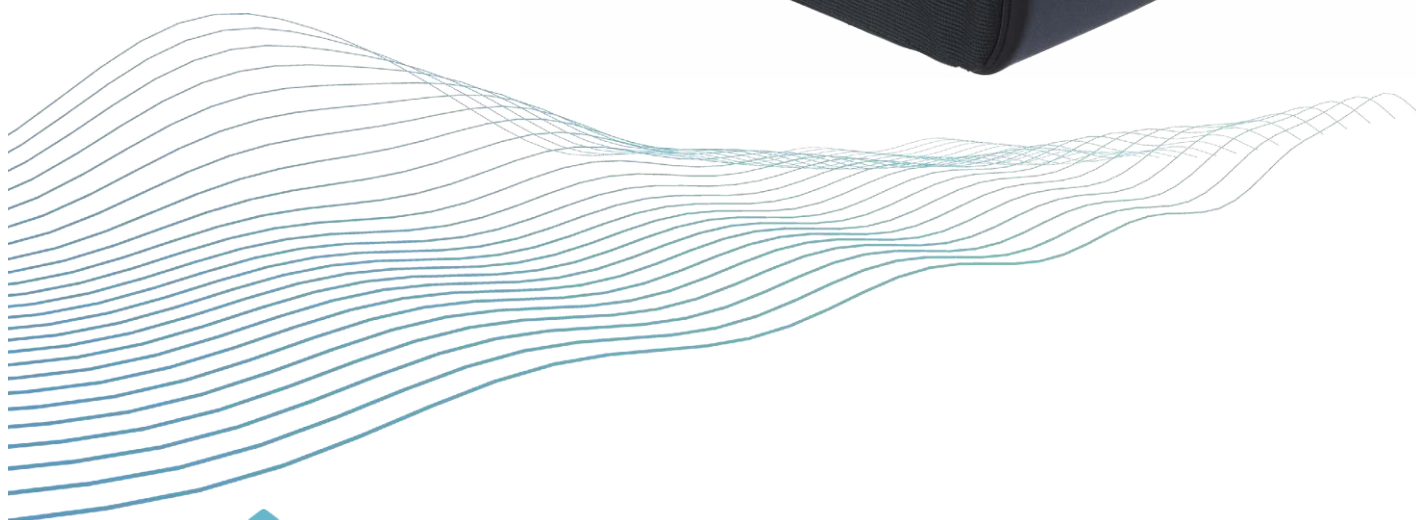TARTA®
by Orthosanit Diffusion

GMN CODE: 805600081AIRY 83

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Airy is the line of cushions for users with different postural needs. Its special breathability and microclimate have been designed for the comfort and well-being of the user. The line has three levels of customization, for active users and users with more complex postural needs.

Intended use:

The Airy cushion is used on the seating surface of a wheelchair. The Airy cushion consists of a:

- Shaped anatomical base in foam
- Foam cylinders with different diameters and different densities
- Air cylinders (*only Control model*)
- A breathable lining
- An Incontinence cover is available on request.



The Airy line has three models:

NB: the cushion must always be used with the supplied cover

General Characteristics

The structure of the cushion consists of shaped foam designed to increase its already high breathability. In addition, holes and grooves are also present to facilitate the passage of air vertically and horizontally.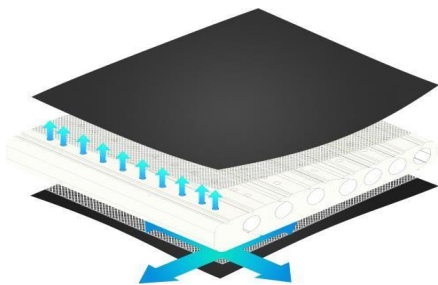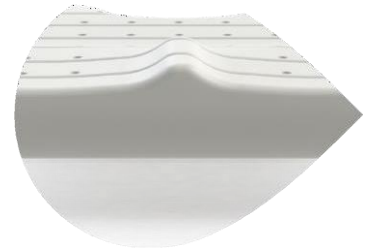

Microclimate system

The cover is made of a breathable fabric and a layer of 3D material designed to further enhance the breathability of the cushion.

The side of the cushion was made to accommodate cylinders composed of foam of different densities or air cylinders. These cylinders are used to adapt the cushion to the different postural needs of the user.

Foam cylinders

Air and foam cylinders

Airy Control Cushion

The Airy Control cushion allows the healthcare professional to customize the posture in the wheelchair inserting different intensities of foam, recognizable by the colour of the cylinder that have different resistance (image A and B). The healthcare professional can decide to leave the section empty to minimize containment.

In the back part of the cushion the foam cylinders used have a diameter of $\varnothing$ 45 mm (marked with the letter A). Instead in the front part the cylinders have a diameter of $\varnothing$ 39 mm (marked with the letter B).

D 80
80 kg/cm³


T 40
40 kg/cm³


T 35
35 kg/cm³

The length of the cylinders go from 180 cm to 230 cm based on the width of the cushion used by the user.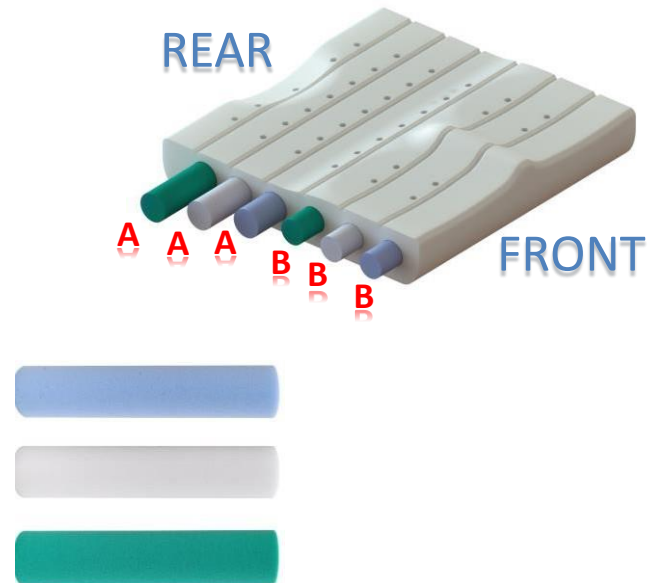

The Airy Control cushion also allows the healthcare professional to use an air insert kit in the back part to customize the correct position of the user. The Airy Control cushion is a system that allows the healthcare professional to use different combinations between the air cylinders and the foam cylinders in the back part. In the front the operator can use the different foam cylinders with different densities.

The user must follow the direction of use of the pillow on the wheelchair (under the pillow you can find the direction of use). The cushion must be used with the cover. It is not recommended to use the pillow during medical treatment of bedsores.

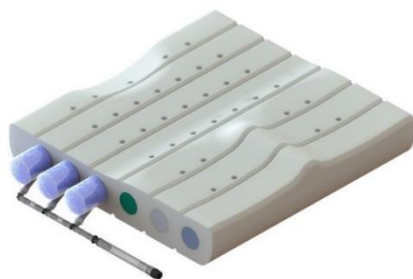The air cylinders allow you to intervene on the posture in an easy, quick and gradual way. The air cylinder kit is made of three self-inflating inserts connected to each other. The inflation system works through the opening and closing of the valve.

The air kit can only be inserted in the ischiatic part of the cushion, independently on the right or left sides.

It is possible to intervene independently on the two air kits to create a pelvic depression or to accommodate asymmetries of the pelvis and much more. The system with air cylinders is only available on the Control cushion.

The different densities foam cylinders allows you to adjust the structure of the cushion according to the postural needs of the user.

You can configure a pre-ischiatic block to reduce forward slipping, create a pelvic depression, fix the asymmetries of the pelvis, create recesses for the femurs and encourage abduction or adduction and much more.

The same pillow can be suitable for different patients.


Adjustment of the air cylinders

The air kit has 3 air cylinders controlled by a special valve with two mechanisms:

The first opens or closes completely the use of the valve;

The second is used to regulate the input or output of the air in the cylinders.
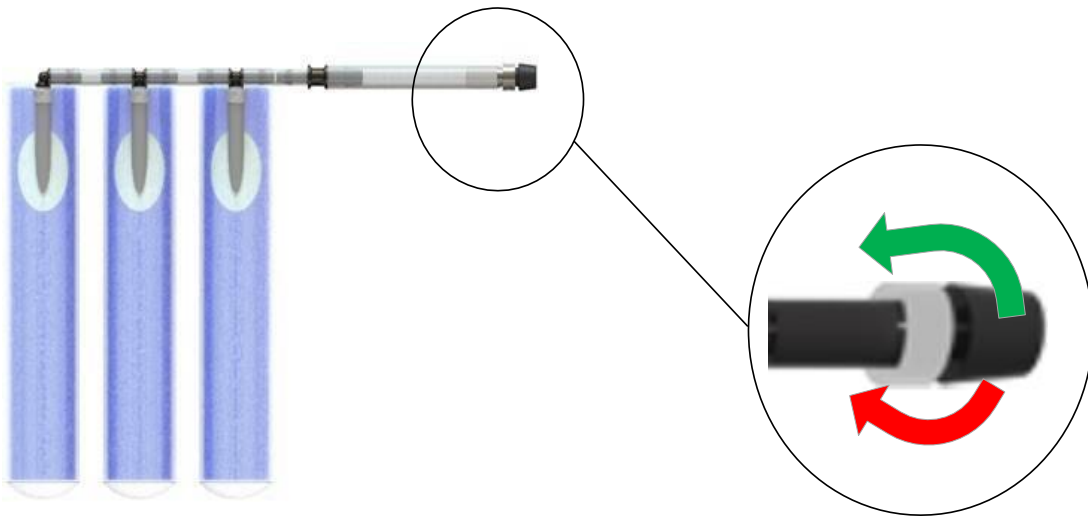


To adjust the inflation of the cylinders press the black nozzle. More the pressure exerted, the higher the air exit speed.

Valve Opening – turning the Grey rack, as indicated by the **Green arrow**, allows the use of the valve to input and output the necessary air for the kit use.

Valve Closure – turning the Grey rack as indicated by the **red arrow** closes the valve.

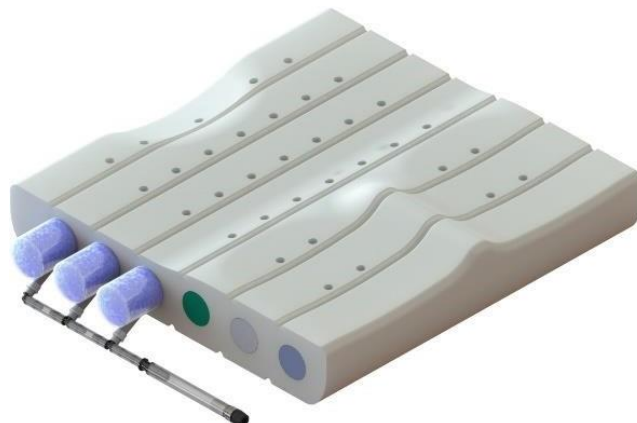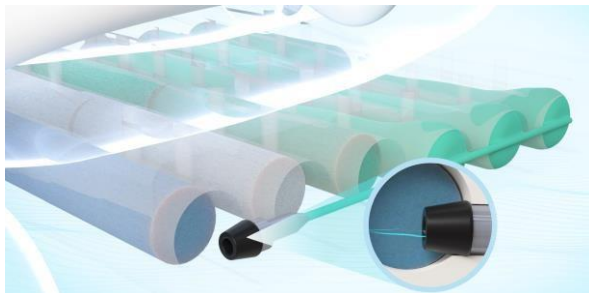Airy Action Cushion

The Airy cushion allows the healthcare professional to contain the deformity through the use of different foam cylinders of different densities, recognizable by the colour that identifies different resistance (image A). The healthcare professional can also decide to leave empty a section to minimize containment.

For the Action cushion it is possible to only use cylinders with a diameter of $\varnothing$ 35 mm both in the back and front of the cushion. The Air cylinders cannot be used on this model.

Soft


Medium


Hard

The length of the cylinders goes from 180 cm to 230 cm based on the width of the cushion used by the user.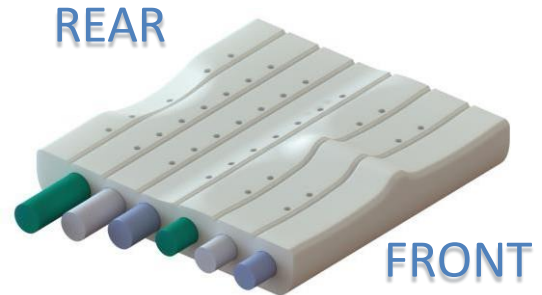


Airy Sport Cushion

The Airy Sport cushion is suggested for very active people. The characteristics of Airy Sport are the lightness, comfort, and especially the concept of the microclimate. The design used for its creation with the holes passing from bottom to top and with the lining is designed to create the right air passage.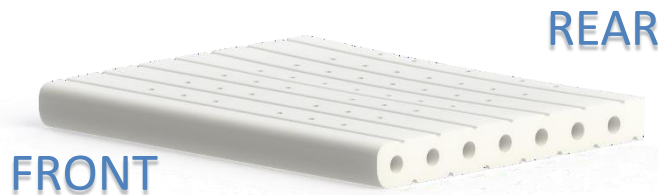


Microclimate control is recognized worldwide as a fundamental factor for the user comfort and the prevention of skin problems including bedsores. The skin must be kept clean and dry. The management of temperature and humidity are important factors for the well-being of the user.

Thermoregulation problems can arise after some diseases such as Multiple Sclerosis, Cerebral Palsy, Parkinson's and spinal injuries. There are also cases where the user remains seated for a long time on passive wheelchairs, where the air circulation is limited by embracing armrests, cushions and backrests. The extremely high temperature of certain areas of the world where in summer the temperatures get really high complicates the management of the body temperature even in situations of physical efficiency and normal mobility. Tarta's answer is a highly breathable cushion designed to contain temperature and humidity.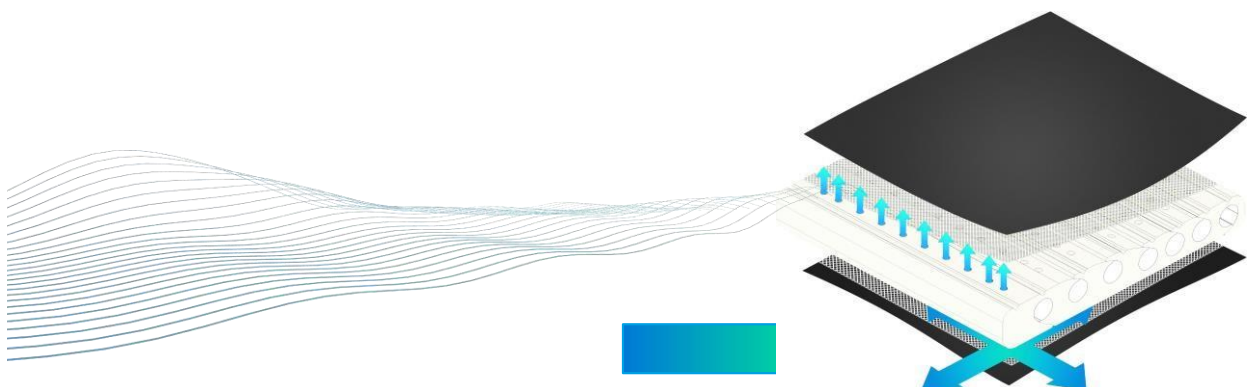

The multiple possibilities offered by the pillow allow the technicians and therapists to explore different solutions until the postural goal is reached.

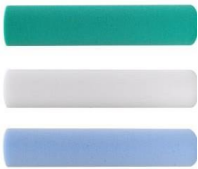Soft

Medium

Hard

BACK

T40	T40
T35	T35
D80	D80
T40	D80
T40	D80
T40	D80

BACK

T40	D80	T40
T40	D80	T40
T40	D80	T40

ABDUCTION

T40	T40
T35	T35
D80	D80
T40	D80
T40	D80
T40	D80

D80	T40	D80
D80	T40	D80
D80	T40	D80

ADDUCTION

T40	T40
T35	T35
D80	D80
D80	D80
T35	
T40	

D80	D80
	T35
	T40

The multiple possibilities offered by the pillow allow the technicians and therapists to explore different solutions until the postural goal is reached.

Soft

Medium

Hard

BACK

T40	T40
T35	T35
D80	D80
D80	D80
T35	T35
T40	T40

BACK

T40	T40
T35	T35
D80	D80
D80	D80
T40	T40
T40	T40

BACK

D80	D80
T35	T35
T40	T40

PRE-ISCIAL BLOCK

T40	T35
T40	T35
D80	T35
D80	D80
T35	T35
T40	T40


T35	T40
T35	T40
T35	D80
D80	D80
T35	T35
T40	T40

	T40
	T35
	D80
D80	D80
T35	T35
T40	T40

D80	D80
T35	T35
T40	T40

T40	
T35	
D80	
D80	D80
T35	T35
T40	T40

PELVIC OBLIQUITY

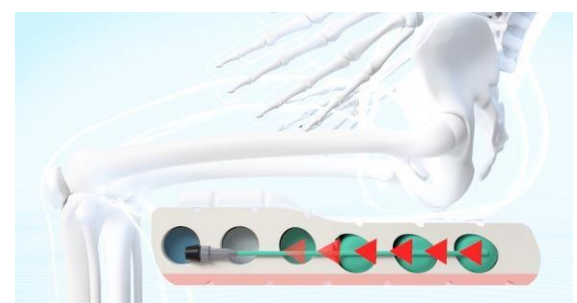INTRODUCTION

The Airy positioning cushion has been designed to provide a high postural support and to adapt to the needs of the body development of individual users. For a proper use of the cushion it is recommended to read these instructions and keep them for future reference.

Maximum capacity:

150 kg for all sizes

WARNING

Using a cushion on a wheelchair can change its centre of gravity by causing it to tip backwards and injuring the user. Before using the cushion on the wheelchair, always adjust the wheelchair or add an anti-tipping wheel to increase stability.

In the first days of using the cushion it's recommended to use it for a short period, it is suggested to try it continuously for a couple of hours and check with your doctor that the skin does not present redness due to pressure. Regularly check that your skin is not reddening. If your skin has redness, stop using the cushion immediately consult a healthcare professional.

WARNING


Avoid exposing the cushion to high temperatures or too cold ones.

Before using the pillow, allow the cushion to warm to room temperature.

Keep your cushion away from sharp objects

Do not insert any obstacle between the pillow and the user, to not reduce the effectiveness of the product.

POSITIONING OF THE CUSHION AND COVER: The product should be used with the correct side positioned upwards. The benefits of the pillow decrease if the cover is not used correctly or it's of an incorrect size resulting in increasing risks of the skin and soft tissues.

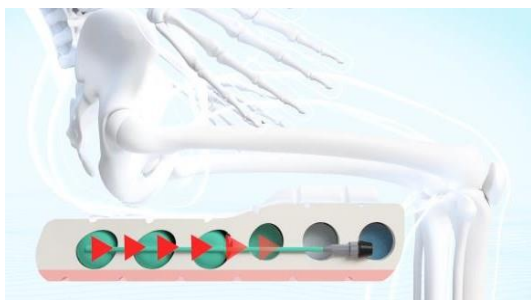NOTE:

It is recommended to use a stable support base, such as the seat of a rigid or folding wheelchair.

Make the user sit on the pillow so that the buttocks are resting on the shaped back and that the legs are comfortably resting on the front raised part.

In addition, the footrest should be adjusted to a low enough position to allow the legs to lean against the front of the pillow.

MAINTENANCE AND CLEANING

- Monthly cleaning and regular maintenance can help extend the life of your pillow. When cleaning it is recommended to examine all parts of the pillow.
- Do not immerse the foam in the water.
- Check that the lining is not torn or excessively worn and replace it if it is broken, cut or not perfectly tidy. Check the correct functioning of the air and foam cylinders.

WASHING SYMBOLS	
	Machine wash in hot water (60 °C)
	If necessary, remove stains using only non-chlorine-based products.
	Air dry
	Do not iron
	Do not dry clean.

Alerts

- Do not use in direct contact with the skin; it is suggested that the user wears clothing to avoid possible allergies and / or skin irritations;
- Skin/soft tissue breakdown can occur due to a number of factors, which vary by individual. Check your skin frequently, at least once a day.
- Redness, bruising or darker areas (compared to normal skin) may indicate the beginning of soft tissue breakdown and may need to be addressed. If there is any discoloration to skin/soft tissue, STOP USE immediately. If the discoloration does not disappear within 60 minutes after disuse, immediately consult a healthcare professional.
- The cushion and the cover MUST be compatible sizes and MUST be used as directed in this manual.
- Do not use your cushion on top of, or in conjunction with, any other products or materials, except as indicated in this manual. Doing so may cause the individual to become unstable and vulnerable to falling.
- Do not use the components of this product separately. The pillow should be used with all components supplied.
- If you are unable to perform any tasks described in this manual, seek assistance.

Precautions

- Do not modify the cushion or any components. Doing so may result in damage to the product and will void the warranty.
- Do not allow your cushion to come into contact with oil-based lotions or lanolin. They may degrade the material.
- Prolonged exposure to high temperatures may degrade materials used in the cushion, affect the performance of the cushion, and void the product warranty.
- Periodically, check the components in case of failure or breakage and replace them if necessary.
- Do not use the control valve to carry or pull the cushion. Carry the cushion from the base or using the proper handle on the cover.

Maintenance

- The individual parts of the device can be cleaned with a soft cloth moistened with water and well wrung out;
- Do not immerse the device in water;
- Do not adjust the device by yourself: if damaged, contact your orthopaedic workshop.
- To remove the cover for replacement, open the zipper of the cover and remove the cushion. After replacing the cover, pay attention to the correct positioning of the cushion. To facilitate this operation, on the base of the cushion you can find the symbol REAR (back) indicating the back of the wheelchair, and FRONT (forward) indicating the front seat of the wheelchair.

Disposal methods

- Some materials and/or components may be recycled; However, the device must be disposed of in line with the methods of disposal of municipal solid waste.

24-month limited warranty

Each Airy pillow is thoroughly checked and tested to offer maximum performance. Each Airy cushion is guaranteed free from manufacturing and material defects for a period of 24 months from the date of purchase, without prejudice to normal use. In the event that a defect in workmanship or material occurs within 24 months from the original date of purchase, Orthosanit Diffusion srl will, at its discretion, repair or replace the product at no cost. This warranty does not cover holes, tears or burns.

For any complaints and repairs, please contact your nearest authorized supplier. Except for the warranties expressly set forth above, all other warranties, including any implied warranties of merchantability and fitness for a particular purpose, are excluded. Warranties outside the above description are excluded. The application of the guarantee is limited to the repair or replacement of goods. Under no circumstances may damage caused by the breakage of products under warranty imply claims for consequential damages or exceed the value of the non-conforming goods sold.

Symbol legend on the label

	CE mark - Complies with EU Directive 2017/745 (MDR)
	This symbol identifies the manufacturer and must be accompanied by the name and address of the manufacturer.
	Device code
	Production batch
	Attention! Before using the device, it is recommended to read the instructions for use.
	Latex-free
	Serial Number
	Class I medical device
	Date of Manufacture

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ORTHOSANIT DIFFUSION S.r.l.